


# Processing Instructions

## Light-curing Base Plates

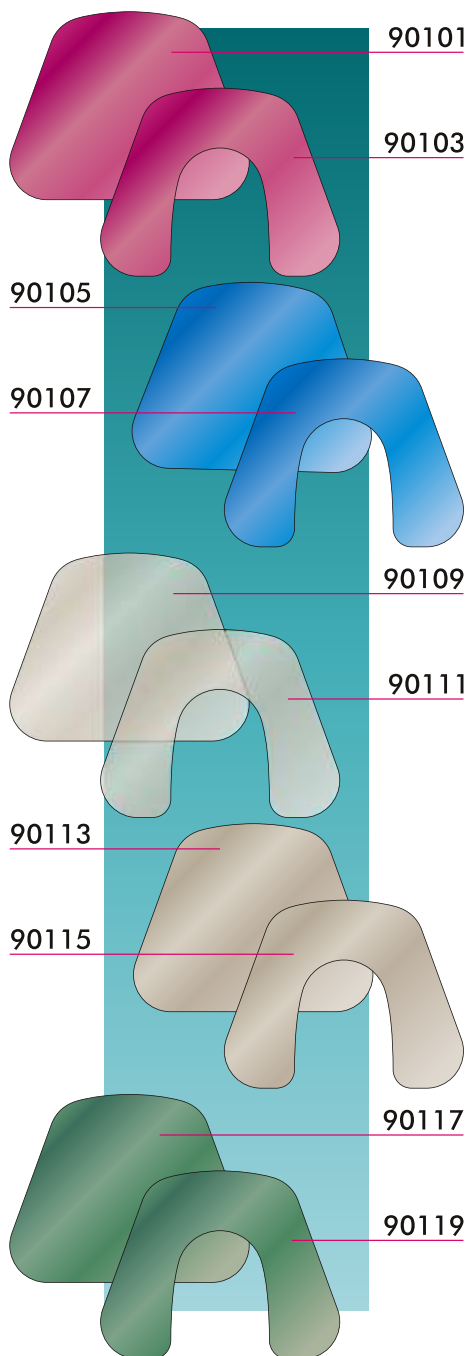

### The Advantages at a Glance

- Quick fabrication of bases and individual trays
- exact fitting after polymerization
- Insulation of plaster models is not required

### Range of Use

- Total and partial prosthetic technique
- individual trays
- bases for bite patterns and odontograms

### Processing Instructions

Mould the light-curing base plates onto the model and cut off the excess material.

Form the material labial/buccal to the jaw and smooth it by hand.

In case of partial models cut out the base palatinal/lingual dependent on the teeth.

The better the modelling is done, the less time for proceeding is required.

Polymerize the plates for around 5 minutes under UVA-light and then finish.

To get back the original shade, apply vaseline or monomer on whitish shining grinding areas.

Store cool (< 20°C) and light-protected.

**vigano**<sup>®</sup>  
DENTAL

Clemensstraße 14  
D 44 789 Bochum  
fon: (++49) (0)234-301784  
fax: (++49) (0)234-330545  
web: [www.vigano-dental.de](http://www.vigano-dental.de)  
email: [info@vigano-dental.de](mailto:info@vigano-dental.de)